


HELPS REDUCE
THE SPREAD OF GERMS



FACE MASK

MASK FITTING INSTRUCTIONS

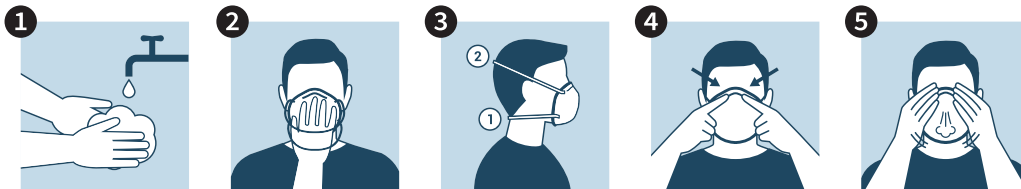

- 1 WASH YOUR HANDS. REMOVE THE DISPOSABLE FACEMASK FROM THE PACKAGING AND UNFOLD IT WITHOUT TEARING
- 2 HOLD THE MASK IN YOUR HAND AND LINE IT UP WITH YOUR EYES, NOSE AND MOUTH. ENSURE THE FORMABLE NOSE PIECE IS AT THE TOP
- 3 PULL THE ELASTICS OVER YOUR HEAD. POSITION THE TOP STRAP ABOVE YOUR EARS AND THE BOTTOM STRAP AT THE BACK OF YOUR HEAD, BELOW YOUR EARS
- 4 MOLD OR PINCH THE FORMABLE NOSE PIECE TO THE SHAPE OF YOUR NOSE
- 5 PERFORM A FIT CHECK. PLACE HANDS OVER THE MASK AND EXHALE, ADJUST IF THERE ARE LEAKS

WARNING: This Fitted Disposable Facemask CANNOT ELIMINATE the risk of contracting infection, illness or disease, including the risk of death. This mask is not approved for medical use. **PROTECTION:** The filtration media in this Fitted Disposable Facemask has been tested under independent laboratory conditions and was challenged with a non-oil, aerosolized *Staphylococcus aureus* spray with a mean particle size of 3.0 +/- 0.3 µm. This media achieved an average bacterial, viral and particulate filtration efficiency rating of 95%. This filtering facemask, when fitted properly to prevent air leakage around the mask perimeter, may reduce the wearers' exposure to airborne biological particles (eg. *Staphylococcus aureus*). **CAUTIONS AND LIMITATIONS:** Wearer must read and understand these User Instructions prior to use: Failure to properly use and maintain this product could result in injury or death. Not to be used in atmospheres containing less than 19.5% oxygen. Not to be used in environments imminently dangerous to life or health. Not to be used for protection from gases, vapours, oil aerosols, asbestos or sandblasting. Not to be used with beards, facial hair or other conditions that may prevent a positive seal between the face and the sealing surface of this facemask. Airborne particulates that could be harmful to your health include particles too small to see. Do not assume that harmful airborne particles are not present if you cannot see them. This facemask is designed for use by adults in occupational or professional settings where proper understanding of its use and limitations is present. This facemask is not designed for use by children. This facemask does not supply oxygen. This facemask can reduce the wearers' exposure to airborne contaminants, however it cannot prevent entry of contaminants through other surfaces such as skin and eyes which would require other means of personal protective equipment. Do not alter, misuse or abuse this facemask. Individuals with a compromised immune or respiratory system such as asthma or emphysema, should consult a physician prior to using this facemask. If breathing becomes difficult while wearing this facemask, leave the contaminated area immediately, ensure you are in an area with non-contaminated air and remove and discard the facemask. If this facemask shows signs of damage, including damage to the elastic straps or nose piece or is soiled, discard and do not use. Always perform a user fit test to ensure a proper mask to face seal in accordance with the mask fitting instructions prior to entering a contaminated area.

 DISPOSE OF THOUGHTFULLY

FOR NON-MEDICAL USE ONLY

Manufactured by:
Industrial Filter Manufacturing Limited
187 Robert Street East
Penetanguishene, ON, Canada L9M 1G9
TF: 1.800.225.9330
E: ifm@liquidfiltration.com
Web: liquidfiltration.com



FOR TECHNICAL ASSISTANCE CALL: 1.800.225.9330